


SECONDARY CONTROL SYSTEM

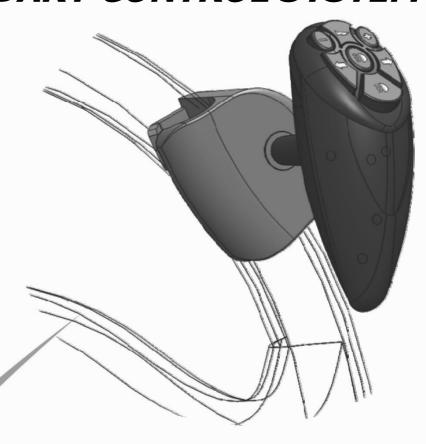

User manual

LEHMDIS

04/04/2014



BAS (NW) Ltd Mobility Controls

School Lane - Hollins Green Warrington - WA3 6LJ Tel: 0161 776 1594 www.basnw.co.uk office@basnw.co.uk



User manual LEHMDIS





Table of contents

USE OF LEHMDIS			
	Presentation	4	
	Power supply Battery reference Replacement of batteries Average replacement period		
	Use and functions		
	Functions not covered		
	Important Activation LEHMDIS functions Cancelling LEHMDIS commands Troubleshooting	- - - -	



Use of LEHMDIS

Presentation

There are 7 function push buttons on the emitter:



* Functions possible depending on vehicle



Power supply

Battery reference


• 1 battery Lithium type CR 123 A

Replacement of batteries

- Remove the screws with slotted/phillips screwdriver
- Pull the bottom of the transmitter
- Change the battery respecting the way

Average replacement period

• 1 year with daily use.

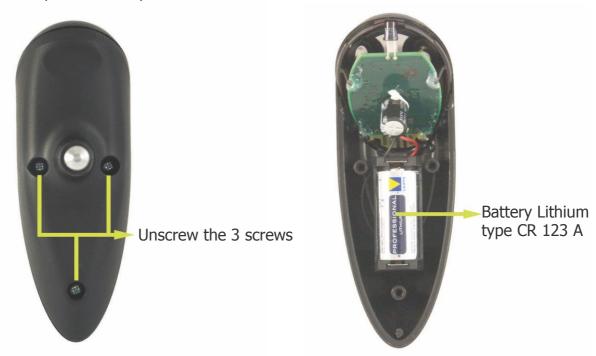



Use and functions

Function		Action	Button
Dight indicator	On	Impulse	$ \Rightarrow $
Right indicator	Off	Impulse	$ \Rightarrow $
Left indicator	On	Impulse	\Diamond
Left indicator	Off	Impulse	\Diamond
Low boom lights	On	Impulse	≣D
Low beam lights	Off	Sustained pressure	 ■D
Lloadiahta	On	Impulse	■D
Headlights	Off	Impulse	■D
Headlights flashing	On	Sustained pressure	D
Horn	Activation	Sustained pressure	\square
Slow or automatic front windshield wiper (depending on vehicle)	On	Impulse	+
Fast front windshield wipers	On	Impulse	+
Front windshield wipers	Off	Impulse	-
Front windshield washers*	On	Sustained pressure	+
Rear windshield washers *	On	Sustained pressure	-

^{*} Functions not for all vehicles

Functions not covered

- Rear windshield wipers

- Side lightsHazard warning lightsFront and rear fog lights



Important

Activation LEHMDIS functions

• If a function was activated with LEHMDIS it is imperative to deactivate with LEHMDIS as well.

Cancelling LEHMDIS commands

• To cancel LEHMDIS commands switch off the engine.

Troubleshooting

• If a button gets stuck, the transmitter stops its transmission after 25 seconds. The other buttons are no longer usable.



Date of the last batteries replacement	
	_
	_
	\dashv

In the Event of an Accident

Should your vehicle be involved in an accident your controls should be checked by the fitter before use.

Warranty

BAS Ltd guarantees this product and installation for three years providing that it is in no way tampered with. All warranty work is done by us at our premises. Warranty is null and void if the product is not serviced annually.

Service Schedule

The first three services are necessary to the validity of your warranty.

First service at 12 months	Date
Second service at 24 months	Date
Third service at 36 months	Date
Fourth service at 48 months	Date
Fifth service at 60 months	Date
These services are validated by our	company stamp



BAS (NW) Ltd Mobility Controls
School Lane - Hollins Green
Warrington - WA3 6LJ
Tel: 0161 776 1594
www.basnw.co.uk
office@basnw.co.uk